





# Evac 1-2-3

## Patient Evacuation System

### Disasters Can Happen Anytime, Anywhere

It's not if, but when. Both natural and man-made events may force the evacuation of patients to alternative facilities with little or no warning. An effective evacuation plan must include an easy-to-use process that operates effectively in the worst of conditions and on a moment's notice.

Evac 1-2-3™. Effective Evacuation by Design.



### Integrated System for Effective Evacuation

Prioritizing patients and moving them from rooms is just part of the evacuation process. A good plan provides for effective patient tracking through all aspects of the evacuation process including staging, transportation and even arrival at definitive care locations. Evac 1-2-3™ is based on lessons-learned and provides all these capabilities in an integrated, intuitive and easy-to-use system that requires minimal training. Each of the three components that make up Evac 1-2-3™ was specifically designed to minimize personnel resources needed to maintain documentation so more effort can be dedicated to expediting evacuation and patient care.



### Keeping Every Patient Evacuation-Ready

Proper preparation is the key to effective evacuation. The Evac 1-2-3™ system starts at patient registration. Upon admittance, an evacuation tag is placed into the patient's chart. Standard patient labels may be applied to the tag for optimal preparedness or preprinted labels may be included in the patient's chart for rapid application to the tag in the event an evacuation is ordered.

### Patient Medical Record Number Association


Unlike other evacuation systems that use random unrelated tag numbers, Evac 1-2-3™ uses the patient's hospital-generated medical record number to ensure accurate tracking. The tag is also labeled with the originating facility to ensure any destination point knows where the patient came from. Utilizing patient labels also allows bar-coding tracking systems to read the tags.

Front side of the Evac 1-2-3™ evacuation tag



# Evacuation Simplified

Evac 1-2-3™ utilizes a systematic tool-set to streamline and simplify the evacuation process while maintaining effective patient tracking and resource allocation for the duration of the event. Fully compliant with HICS documentation requirements, EVAC 1-2-3™ not only optimizes patient care but also ensure all aspects of the evacuation are documented with minimal effort.

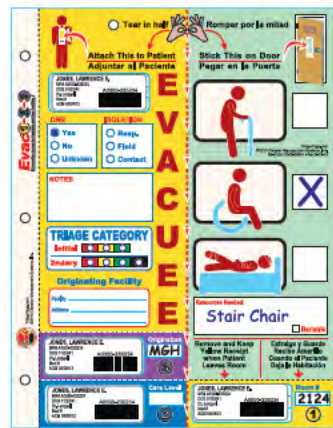

## Evacuation Preparation

Once removed from the patient chart, the evacuation tag is separated along its center perforation. One half is placed on the patient and the other on the room door. The resource requirements to move the patient are indicated on the door section for clear identification to staff.



## Patient Evacuation

As patients are moved out of their room the door section's receipt is removed and placed in the Evac1-2-3™ receipt holder. The door label now indicates the patient has left the room. The receipt holder provides an effective patient tracking tool to maintain a record of continuity-of-care for the patients moved by a specific staff member.



## Staging and Transportation

As patients wearing the evacuation tag are moved to the staging area, vital information such as triage acuity and SBAR comments may be indicated on the tag as part of the transport prioritization and resource requirement process. During the transport loading phase, the tag's patient transportation receipt is removed and placed in the patient evacuation tracking receipt holder. This provides a chronological record of each patient transported to specific locations.



## Destination Arrival

As patients arrive at a destination location, the last receipt is removed from the patient's tag and placed in the destination receipt holder. Evac 1-2-3™ provides a complete documentation trail for full patient accountability and tracking.



**Learn more**  
Scan the code to watch the video.





## Tag Key Features

**Bilingual and symbolic**

**Patient mobility indicators**

**SBAR documentation**

**Room evacuation receipt**

**Remove and Keep Yellow Receipt when Patient Leaves Room**  
**Extraiga y Guarde Recibo Amarillo Cuando el Paciente Deja la Habitación**

**Attn: Evacuation Personnel Tear-Off & Retain This Receipt as Patient is Evacuated from Room**

**Attn: Transportation Officer Tear-Off & Retain This Receipt as Patient Boards for Destination**


DNR and ISO indicators

Triage assessments

Facility identifier

Destination receipt


Transport receipt

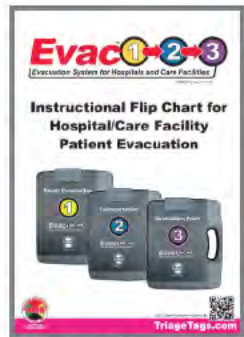

Evac 1-2-3™  
instructional flip-chart  
DMS-05879

## System Configurations

In addition to the clipboard kits, Evac 1-2-3™ allows you to order components based off your facilities evacuation needs, bed count and destination facilities.

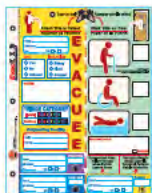

### Evacuation tag

Provided in packs of 20. If used with the patient label holder, order enough tags for each bed in your facility.  
DMS-05854



### Patient label holder

Used in the patient's chart to allow for the evacuation tags to be quickly labeled during an evacuation.  
DMS-05934




### Evacuation receipt holder

Each holder can hold up to 10 patient evacuation receipts. Sold in packs of 5.  
DMS-05855



### Transportation receipt holder

HICS compliant transportation receipt holder tracks 10 patients per sheet. Sold in packs of 5.  
DMS-05858

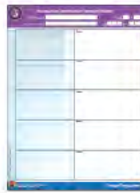

### Destination receipt holder

Each receipt holder logs-in 10 arriving patients. Consider issuing these to service providers as well as know destination facilities. Sold in packs of 5.  
DMS-05859



## Position Kits

### 1 Room Evacuation



Placed strategically throughout the facility, each Room Evacuation Kit includes 20 evacuation tags and 2 receipt holders to track 20 patients.  
DMS-05880

### 1 Room Evacuation Receipt Holder

Originating Facility: **MGH** Nurse ID: **RN283** Page **1** of **2**  
 Incident Name: **Hurricane Stella** Date: **07/07/14** Time: **15:30**

Disposition:  Home  Transfer

ALS  BLS  CCT  NICU  Bus/Van

Notes:  
O2 onboard.  
Pt. brought to staging.

ORTIZ, DENNIS W.  
MRN A0000-236255  
DOB 10/15/61  
R. Farmstead  
Med # ADM070112

Room # **2130**

### 2 Transportation



Used in the patient loading area, the HICS compliant Transportation Kit can document the transportation of up to 200 patients.  
DMS-05860

### 2 HICS 255 Master Patient Evacuation Tracking & Transportation Receipt Holder

Originating Facility: **MGH** Date: **07/07/14** Time: **15:30**  
 Incident Name: **Hurricane Stella** Patient Tracking Manager: **J. Hunter**

Disposition:  Home  Transfer

ALS  BLS  CCT  NICU  Bus/Van

Destination: **MGH** Arrival Confirmed:   
 Time Destination Contacted and Report Given: **15:45**  
 Transfer Time: **16:00**  
 Sent Via: **AMB 251**  
 Admission Location: **4th Flr ICU**  
 Medication Sent:  Family Notified:  Medical Records Sent:

ORTIZ, DENNIS W.  
MRN A0000-236255  
DOB 10/15/61  
R. Farmstead  
Med # ADM070112

Care Level: **ICU**

Triage Category: **IMMEDIATE**  
 DELAYED  
 MINOR  
 EXPIRED

### 3 Destination



Used by the destination facility, the Destination Kit can document the receiving of 200 patients.  
DMS-05861

### 3 Evacuation Destination Receipt Holder

Incident Name: **Hurricane Stella** Page **5** of **8**  
 Recorder Name: **C. Fleisher** Date: **07/07/14** Time: **17:45**

Notes:  
10:42 Pt. arrived. Sent to admittance.  
Reunited with Husband

ORTIZ, DENNIS W.  
MRN A0000-236255  
DOB 10/15/61  
R. Farmstead  
Med # ADM070112

Origination: **MGH**



**Disaster Management Systems**  
www.TriageTags.com  
866.565.7597

©2013 Disaster Management Systems, Inc.